

FUJIFILM

Cleaning and Disinfection guide
An excerpt of the operation manual

MOBILE DIGITAL X-RAY SYSTEM DR-XD 1000

FDR nano

(MOBILE X-RAY UNIT DR-XD 1000PX)

Cautions:

This is for reference use only.

The contents of this guide is subject to change without prior notices.

Disinfection and Cleaning Instructions



WARNING

- Confirm that the respiratory density of disinfectant including solvent is under legal regulation. Certain disinfectants may damage health. When using a disinfectant, follow instructions supplied by the manufacturers.
- Do not use the following disinfectants or sterilizers at the time of disinfection. Quality, performance and safety of each device cannot be assured.
 - Chloric disinfectant which is strongly corrosive to metals and rubber parts.
 - Disinfectant whose uses on metals, plastics, rubber parts and coating are forbidden according to the instructions supplied with the disinfectant.
 - Formalin gas and disinfectant sprays that may get inside the equipment.
Disinfectant ethanol is recommended for disinfection. Carefully read the instructions and cautions supplied with the disinfectant before use. For details on the disinfectant, contact a FUJIFILM dealer or the service representatives at the agency from which you purchased the disinfectant.



CAUTIONS

If the sensor unit of the flat panel sensor is not disinfected, it may lead to secondary infection. Clean the sensor unit of the flat panel sensor with ethanol for disinfection, etc. for each patient to prevent infection.