

**List of disinfectants that can be used**

Below cleaners and disinfectants have been tested compatibility to use on our products. Usage of other products shall be done under the full responsibility of the user.

Area for cleaning and disinfection	Usable cleaning and disinfecting products on X-ray imaging devices
Surface of products	<p>Disinfectant ethanol (ethanol of 70 to 75 wt% or 76.9 to 81.4 vol% without other additives) can be used for below products.</p> <ul style="list-style-type: none"> <li>- FDR D-EVO II</li> <li>- FDR nano</li> <li>- FDR Xair</li> <li>- AMULET Innovality</li> </ul> <p>Sodium hypochlorite (0.5% or less dilution)</p> <ul style="list-style-type: none"> <li>- FDR D-EVO II</li> <li>- FDR nano</li> <li>- AMULET Innovality</li> </ul> <p>Sodium hypochlorite cannot be used on metal materials as it could cause corrosion.</p>