

# FUJI HUNT

FUJIFILM

# X-RAY CHEMICAL SYSTEM

FOR THE MANUAL  
PROCESSING OF ALL  
MEDICAL X-RAY FILMS.

FUJI HUNT PHOTOGRAPHIC  
CHEMICALS, PTE. LTD



## SYSTEM DESCRIPTION

XD-25 Developer is a single all-liquid concentrate designed for processing a wide range of medical X-ray emulsions. It provides consistent radiographic quality for an extended period of time under normal manual processing conditions. Even when processing a few films per day, XD-25 Developer will maintain excellent film contrast levels in the working solution.

XF-25 Fixer is recommended as the companion product.

PRODUCT	CATALOG NO.
XD-25 Developer (4 x 25L)	873460
XF-25 Fixer (4 x 25 L)	873500

## DIRECTION FOR USE

The time of development should be varied to suit the type of films utilized and the requirements of the individual user. However, a development time of 3 to 5 minutes at 20°C is recommended as a starting point.

## HANDLING PROCESSING SOLUTIONS

All photographic processing solutions can exert harmful effects when brought into contact with human tissue to a greater or lesser extent depending on the nature of the solution and its concentration. All users of such solutions should exercise the greatest care to avoid the chemicals contacting the skin, eyes or other parts of the body. Always wear solution resistant gloves and effective eye protection. In case of accidental contact with processing solutions wash the affected part with plenty of clean cold running water. Wash with an acidic soap and rinse thoroughly with water. Consult a medical doctor.




## DIRECTIONS FOR USE

**XD-25 Developer:** Dilute one part concentrate with four parts water. (1:4)


**XF-25 Fixer:** Dilute one part concentrate with four parts water. (1:4)

### MIXING INSTRUCTIONS

#### X RAY DEVELOPER WORKING TANK SOLUTION

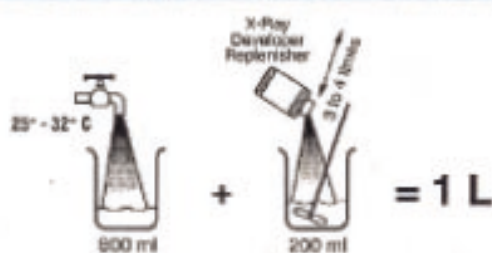

#### X RAY FIXER WORKING TANK SOLUTION

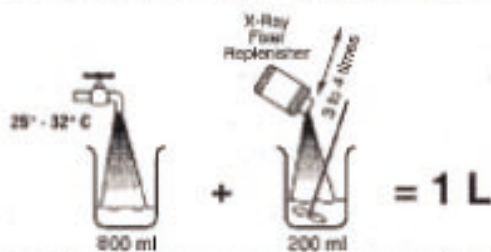

# FUJI HUNT

FUJIFILM

**FUJI HUNT PHOTOGRAPHIC CHEMICALS, PTE. LTD.**

RCB Reg No : 198901418E

15 TUAS AVENUE 7, SINGAPORE 639270.

TEL: 8622116 FAX: (65) 8614829

The data contained in this photographic bulletin is believed to be true and accurate, but is offered solely for your consideration, investigation and verification. Nothing herein shall be construed to be a warranty or guarantee by either Fuji Hunt Photographic Chemicals, Inc. (FUJI HUNT) or any of its affiliates, and all such warranties, implied or otherwise, including and implied warranty of merchantability, are hereby expressly disclaimed. You, of course, are fully responsible for any use and/or domestic or foreign sale of the product(s) described, and nothing in this photographic bulletin shall be construed to constitute permission or a recommendation to practice any invention covered by a patent or a patent application or know-how owned by Fuji Hunt, its affiliates or others.

MANUAL X-RAY 02/96/00