

Cleaning, Disinfection and Sterilization list for Ultrasound System

This list shows the disinfectants that can be used in the system of Ultrasound system. The explanation of C, N and U in the list is as follows.



Item	Definition
C:	Compatible
C*:	Compatible. Soak the cutting cotton or gauze with alcohol and squeeze hard, and then wipe it lightly.
C*2:	Compatible. Do not wipe it strongly.
C*3:	Compatible. Soak with a small amount of liquid in the cloth and then wipe it lightly.
N:	No (do not use)
U:	Unknown/Untested



CAUTION

The disinfectants listed in this list guarantee resistance to the system. However, "cleaning / disinfecting / sterilizing effects" is not guaranteed. For "cleaning / disinfecting / sterilizing effects", please refer to each disinfectants manufacturer's website, instruction manual, etc.

- Each parts of system

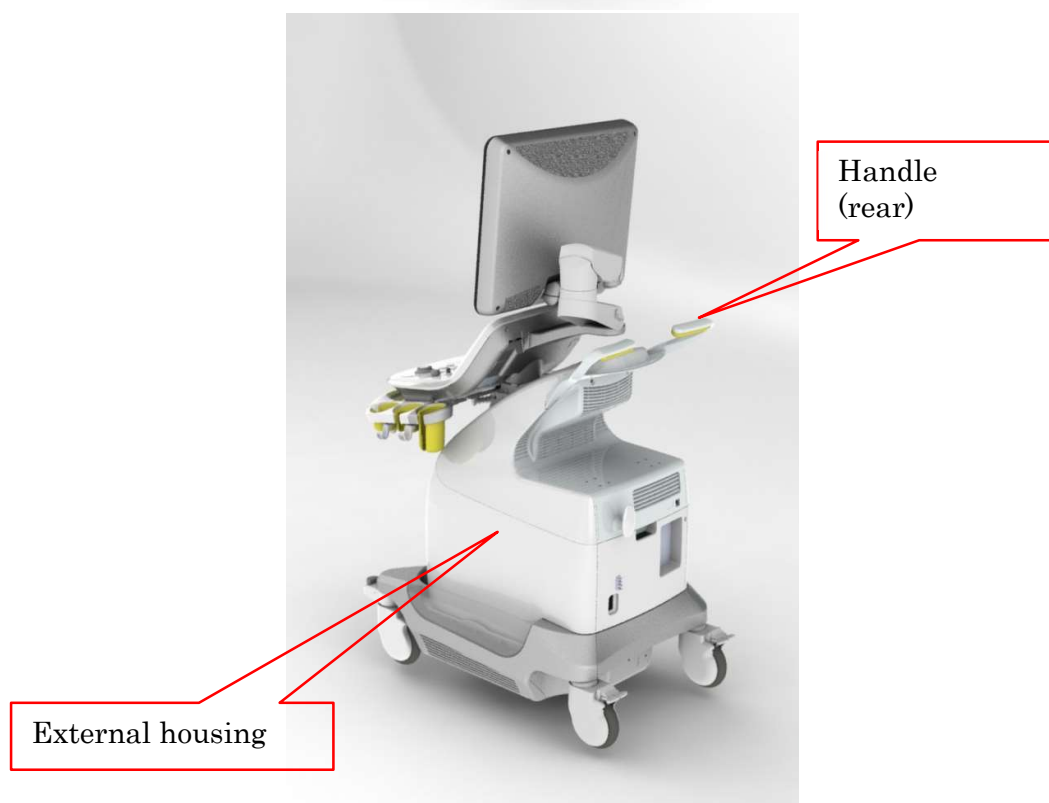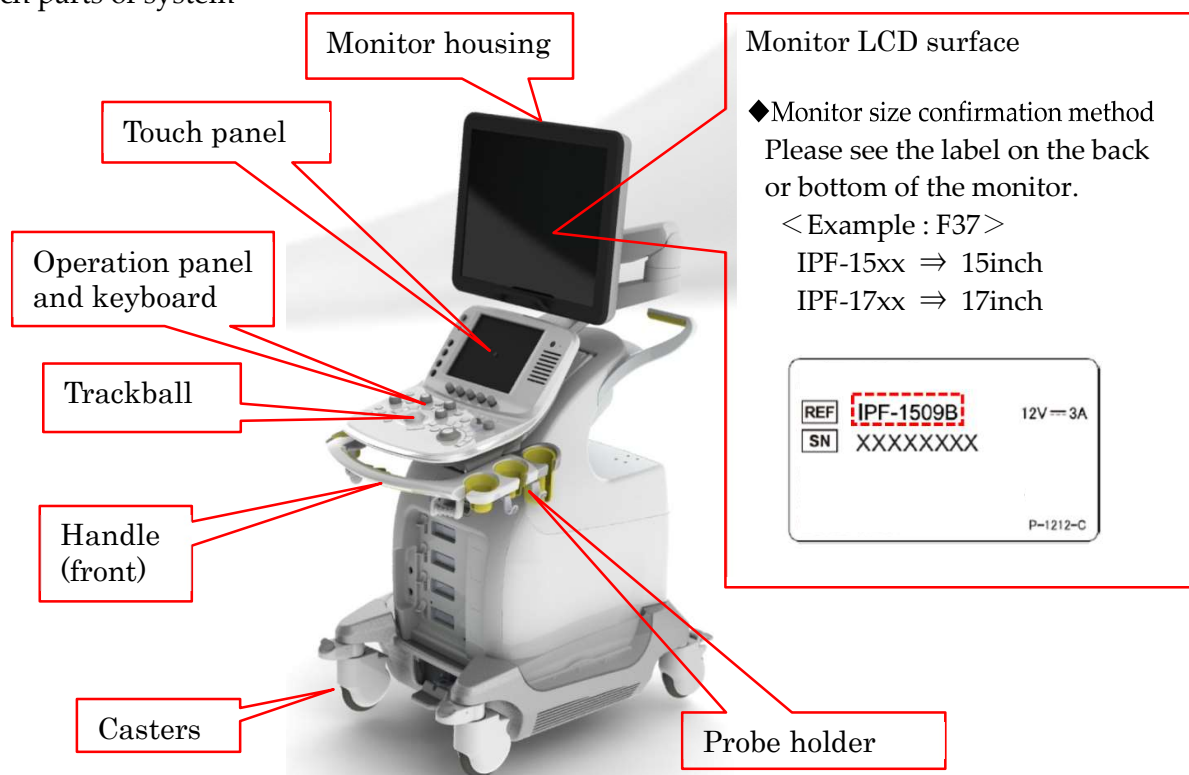

The product names, etc. described in this material are registered trademarks or trademarks of their respective companies.

FUJIFILM Corporation

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	Prosound Alpha6								Prosound Alpha7										
						15 Inch monitor LCD surface	17 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	15 Inch monitor LCD surface	17 Inch monitor LCD surface	19 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	N	C	C	C	C	C	C	C	C	N	C	C*2	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	N	C	C	C	C	C	C	C	C	N	C	C*2	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	N	C	C	C	C	C	C	C	C	N	C	C*2	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	U	C	C	C	C	C	C	C	U	U	U	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	N	C	C	C	C	C	C	C	C	N	C	C*	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	N	C	C	C	C	C	C	C	C	N	C	C*	C	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	Prosound F75							
						19 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	C	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C
セイフキーブ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	F37										F31									
						15 Inch monitor LCD surface (IPF-1507)	15 Inch monitor LCD surface (IPF-1507B)	17 Inch monitor LCD surface (IPF-1707)	17 Inch monitor LCD surface (IPF-1707E)	Monitor housing	Operation panel and keyboard	Probe holder	Handle	Casters	External housing	15 Inch monitor LCD surface (IPF-1509)	15 Inch monitor LCD surface (IPF-1509B)	Monitor housing	Operation panel and keyboard	Probe holder	Handle	Casters	External housing		
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	U	N	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	U	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	U	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyltrimethylammonium chloride	U	N	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	U	N	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	C	U	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	C	U	U	U	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	U	N	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	U	N	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	ARIETTA 60												ARIETTA V60											
						17 Inch monitor LCD surface (IPF-1703B)	21 Inch monitor LCD surface (IPF-2101B)	21 Inch monitor LCD surface (IPF-2101E/F)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	17 Inch monitor LCD surface (IPF-1703B)	21 Inch monitor LCD surface (IPF-2101B)	21 Inch monitor LCD surface (IPF-2101E/F)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	17 Inch monitor LCD surface (IPF-1703B)	21 Inch monitor LCD surface (IPF-2101B)	21 Inch monitor LCD surface (IPF-2101E/F)	Monitor housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	U	U	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	U	U	C	C
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	U	U	C	C
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyl dimethyl ammonium chloride	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	U	U	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	U	U	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	C	U	C	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	U	U	C	C
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	U	U	C	C
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	U	C	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	U	U	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	U	U	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	U	U	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	ARIETTA S60												ARIETTA 70													
						17 Inch monitor LCD surface (IPF-1703B)	21 Inch monitor LCD surface (IPF-2101B)	21 Inch monitor LCD surface (IPF-2101E/F)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	21 Inch monitor LCD surface (IPF-2101B)	21 Inch monitor LCD surface (IPF-2101E/F)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing							
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	C	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	C	U	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	ARIETTA V70										ARIETTA S70									
						21 Inch monitor LCD surface (IPF-2101B)	21 Inch monitor LCD surface (IPF-2101E/F)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	21 Inch monitor LCD surface (IPF-2101B)	21 Inch monitor LCD surface (IPF-2101E/F)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing		
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecylmethylammonium chloride	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
サラヤ環境拭拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C	C	
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	C	C	C	C	C	U	C	C	C	C	C	C	C	C	C	
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	ARIETTA Precision								ARIETTA Prologue					
						21.5 Inch monitor LCD surface (back end unit)	11.6 Inch monitor LCD surface (remote controller)	Monitor housing	Probe holder	Handle	Casters	External housing	11.6 Inch monitor LCD surface (Back end unit)	Monitor housing	Probe holder	Handle	Casters	External housing	
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	C	C	C	C	C	C	C	C	C	C	C	C	C	
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	C	C	C	C	C	C	C	C	C	C	C	C	C	
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	C	C	C	C	C	C	C	C	C	C	C	C	C	
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	C	C	C	C	C	C	C	C	C	C	C	C	C	
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C	C	C	C	C	C	
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecylidimethylammonium chloride	C	C	C	C	C	C	C	C	C	C	C	C	C	
Mikrozyd Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C	C	C	C	C	C	
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	C	C	C	C	C	C	C	C	C	C	C	C	C	
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C*2	
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	C	C	C	U	C	C	C	C	C	
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	ALOKA ARIETTA 850										ALOKA LISEND 880									
						22 Inch monitor Organic EL surface(IPF-2201)	22 Inch monitor Organic EL surface(IPF-2201B)	23 Inch monitor LCD surface(IPF-2301)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	22 Inch monitor Organic EL surface(IPF-2201)	22 Inch monitor Organic EL surface(IPF-2201B)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Mikrozyd Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	ARIETTA750										ARIETTA750 LE									
						22 Inch monitor Organic EL surface (IPF-2201B)	23 Inch monitor LCD surface (IPF-2301)	21.5 Inch monitor LCD surface (IPF-2105)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	22 Inch monitor Organic EL surface (IPF-2201B)	23 Inch monitor LCD surface (IPF-2301)	21.5 Inch monitor LCD surface (IPF-2105)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecylmethylammonium chloride	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrozyd Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	ARIETTA750 SE								ARIETTA750 VE							
						23 Inch monitor LCD surface(IPF-2301)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	21.5 Inch monitor LCD surface(IPF-2105)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	FUTUS						
						21.5 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder bottom	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	C	C	C	C	C	C	C
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	C	C	C	C	C	C	C
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	ARIETTA650										ARIETTA65									
						22 Inch monitor Organic EL surface(IPF-2201B)	21.5 Inch monitor LCD surface(IPF-2104)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	21.5 Inch monitor LCD surface(IPF-2104)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing			
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didcyl dimethylammonium chloride	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	ARIETTA50							
						21.5 inch monitor LCD surface (IPF-2104-FD)	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	C	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	C	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	C	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	C	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	C	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	C	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	C	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	C	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecylidimethylammonium chloride	C	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	C	C	C	C	C	C	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	C	C	C	C	C	C	C	C
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	C	C	C	N	C	C	C	C
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	C	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	HI VISION Ascendus								HI VISION Preirus								
						19 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Probe holder	Handle (front)	Handle (rear)	Casters	External housing	17 Inch monitor LCD surface	19 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Probe holder	Handle (front)	Handle (rear)	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	C*2	C	C	C	C	C	C	C	C	C*2	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	C*2	C	C	C	C	C	C	C	C	C*2	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C*2	C	C	C	C	C	C	C	C	C*2	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	C*2	C	C	C	C	C	C	C	C	C*2	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C*2	C	C	C	C	C	C	C	C	C*2	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
サラヤ環境清拭クロス (サラヤ)	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	C	C	C	C	U	U	C	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C*	C	C	C	C*	C	C	C	C	C*	C	C	C	C*	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C*	C	C	C	C*	C	C	C	C	C*	C	C	C	C*	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	HI VISION Avius							Noblus						
						17 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Probe holder	Handle	Casters	External housing	15 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	C	C	C	U	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecylmethylammonium chloride	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	C	C	C	C	C	C	C	C	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	C	C	C	C	C	C	C
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	C	C*2	C*2	C	C	C	C*2
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	C	C	C	C	C	C	C	C	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	SSD-900								SSD-1000									
						Monitor surface	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Probe holder cover	Probe holder	Handle	External housing	15 Inch monitor CRT surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
セイフキーブ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	U	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	U	C	U	U	C	C	C	C	U	C	C	C	U	U	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	SSD-3500											SSD-4000										
						Monitor surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Probe holder	Handle	Casters	External housing	15 Inch monitor CRT surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Operation panel and keyboard dial	Probe holder cover	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
サラヤ環境拭拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	U	U	U	C	C	C	U	C	C	C	U	U	U	C	U	U	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	U	C	C	C	U	C	C	C	C	C	U	C	C	C	U	U	C	C	C	C*	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	U	C	C	C	U	C	C	C	C	C	U	C	C	C	U	U	C	C	C	C*	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	SSD-5000												SSD-5500											
						15 Inch monitor CRT surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Touch panel	Probe holder	Handle	Casters	External housing	15 Inch monitor CRT surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Touch panel	Probe holder	Handle	Casters	External housing		
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N		
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N		
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyl dimethyl ammonium chloride	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
サラヤ環境拭拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U		
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U		
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	U	U	U	C	U	C	C	U	C	C	C	U	U	U	C	U	C	C		
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	U	C	C	C	U	U	U	C	C*	C	C	U	C	C	C	U	U	U	C	C*	C	C		
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	U	C	C	C	U	U	U	C	C*	C	C	U	C	C	C	U	U	U	C	C*	C	C		

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	SSD-Alpha5										SSD-Alpha10									
						Monitor LCD surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Probe holder cover	Probe holder	Handle	Casters	External housing	Monitor surface	Monitor housing	Operation panel and keyboard	Touch panel	Probe holder	Handle	Casters	External housing	
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
サラヤ環境拭拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	U	U	C	U	U	C	C	U	C	C	U	C	U	C	C	
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	U	C	C	C	U	U	C	C	C*	C	C	U	C	C	U	C	C*	C	C	
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	U	C	C	C	U	U	C	C	C*	C	C	U	C	C	U	C	C*	C	C	

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	Prosound 2								Prosound 6										
						Monitor LCD surface	Operation panel	Rubber key	Trackball (case)	Trackball (ball)	Probe holder cover	Probe holder	Handle	External housing	12 Inch monitor LCD surface	Monitor housing	Operation panel and keyboard	Trackball (case)	Trackball (ball)	Trackball (ball cover)	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	U	U	U	C	U	C	C	U	C	C	C	U	U	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	U	C	C	U	U	C	C	C	C	U	C	C	C	U	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	U	C	C	U	U	C	C	C	C	U	C	C	C	U	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	EUB-5500							EUB-6500						
						Monitor surface	Monitor housing	Operation panel and keyboard	Probe holder	Handle	Casters	External housing	Monitor surface	Monitor housing	Operation panel and keyboard	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	C	C	C	U	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyldimethylammonium chloride	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Mikrozyd Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	U	C	C	C	C	C	C	U	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	U	C	C	C	C	C	C	U	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	EUB-7500							EUB-8500						
						Monitor surface	Monitor housing	Operation panel and keyboard	Probe holder	Handle	Casters	External housing	Monitor surface	Monitor housing	Operation panel and keyboard	Probe holder	Handle	Casters	External housing
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	C	C	C	U	C	C	C	C	C	C
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	N	N	N	N	N	N
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecylmethylammonium chloride	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	U	C	C	C	C	C	C	U	C	C	C	C	C	C
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
クリンクリン II	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U	U
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	U	C	C	C	C	C	C	U	C	C	C	C	C	C
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	U	C	C	C	C	C	C	U	C	C	C	C	C	C

Product name and general name	Country of manufacturer	Manufacturer	Type	Inteded use	Active ingredient	HI VISION 900							OPTION					
						Monitor surface	Monitor housing	Operation panel and keyboard	Probe holder	Handle	Casters	External housing	Probe holder (MP-PH8)	ECG Cable	ECG Clip	PCG Cable	Pulse Cable	ECG Cable for FLUTUS
Sani-Cloth® HB	USA	PDI	Wipe	disinfection	Quaternary ammonium (alcohol free)	U	C	C	C	C	C	C	C	U	U	U	U	U
Sani-Cloth® Plus	USA	PDI	Wipe	disinfection	Quaternary ammonium (low alcohol)	U	C	C	C	C	C	C	C	U	U	U	U	U
Super Sani-Cloth®	USA	PDI	Wipe	disinfection	Isopropanol	U	C	C	C	C	C	C	C	U	U	U	U	U
SONO ULTRASOUND WIPES	USA	ADVANCED ULTRASOUND SOLUTIONS inc.	Wipe	disinfection	Quaternary ammonium	U	C	C	C	C	C	C	C	U	U	U	U	U
ProteX wipe	USA	Parker Laboratories, inc.	Wipe	disinfection	Tetrasodium ethylenediaminetetraacetate	U	C	C	C	C	C	C	C	U	U	U	U	U
Accel PREvention Wipes	Canada	AHP	Wipe		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C	U	U	U	U	U
Accel PREvention RTU	Canada	AHP	Liquid		Accelerated hydrogen peroxide	U	C	C	C	C	C	C	C	U	U	U	U	U
Cleanisept Wipes	EU	Dr. Schumacher GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	C	U	U	U	U	U
Cleanisept Wipes forte	EU	Dr. Schumacher GmbH	Wipe	cleaning	Ammonium	U	C	C	C	C	C	C	C	U	U	U	U	U
Meliseptol®HBV Tissues	EU	B.Braun Medical AG	Wipe	disinfection	Propanol	U	C	C	C	C	C	C	C	U	U	U	U	U
Mikrobac® Tissues	EU	BODE Chemie GmbH	Wipe	disinfection	Ammonium	U	C	C	C	C	C	C	C	U	U	U	U	U
Tristel Duo	EU	Tristel Company	foam	disinfection	Chlorine dioxide	U	C	C	C	C	C	C	C	U	U	U	U	U
Incidin® OxyWipe S	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	U	U	U	U	U
Incidin® OxyWipe	EU	Ecolab Healthcare	Wipe	disinfection	Hydrogen peroxide	N	N	N	N	N	N	N	N	U	U	U	U	U
WIP'ANIOS EXCEL	EU	Laboratoires ANIOS	Wipe	cleaning	Didecyl dimethyl ammonium chloride	U	C	C	C	C	C	C	C	U	U	U	U	U
Mikrozid Universal Wipes	EU	Schuelke & Mayr GmbH	Wipe	disinfection	Low alcohol	U	C	C	C	C	C	C	U	U	U	U	U	U
ハクゾウ環境クロスVロック	JPN	HAKUZO Medical (ハクゾウメディカル)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	U	U	U	U	U
サラヤ環境清拭クロス	JPN	SARAYA (サラヤ)	Wipe	cleaning	Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	U	U	U	U	U
セイフキープ	JPN	KAO (花王)	Wipe		Quaternary ammonium salt (第4級アンモニウム塩)	U	C	C	C	C	C	C	C	U	U	U	U	U
クリンクリンⅡ	JPN	TOWA CORPORATION (東和コーポレーション)	Wipe		Benzalkonium Chloride (塩化ベンザルコニウム)	U	U	U	U	U	U	U	U	U	U	U	U	U
HydroAG+	JPN	FUJIFILM (富士フイルム株式会社)	Wipe / Liquid	antibacterial	Ethanol	U	U	U	U	U	U	U	U	U	U	U	U	U
Sodium hypochlorite	-		Liquid	disinfection	Sodium hypochlorite (次亜塩素酸ナトリウム)	U	C	C	C	C	C	C	C	U	U	U	U	U
Ethyl alcohol max. 80% vol.	-		Liquid	disinfection	Ethanol	U	C	C	C	C	C	C	C	C	C	U	C	N
Isopropyl alcohol max. 80% vol.	-		Liquid	disinfection	Propanol	U	C	C	C	C	C	C	C	U	U	C	U	N